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Instruction for use

Please read this information carefully because it will help you to use HYLO® FRESH properly.

HYLO® FRESH

Sodium hyaluronate 0.3 mg/ml and Euphrasia-original mother tincture
For relief of irritated eyes

Sterile, phosphate-free eye drops without preservatives

What is HYLO® FRESH?

HYLO® FRESH is a sterile preservative-free solution containing 0.3 mg/ml sodium hyaluronate, Euphrasia-original mother tincture, a borate buffer and water.

What is HYLO® FRESH used for?

For relief of irritated eyes.


When should HYLO® FRESH be used?

HYLO® FRESH is used for the relief of irritated eyes. Irritated and dry eyes can result from environmental stress such as intensive computer-based activity, dry climate, draught, increased solar radiation, air pollution caused by dust or pollen or wearing contact lenses. For hundreds of years, preparations of eyebright (*Euphrasia officinalis*) are traditionally used for the treatment of conjunctival irritation with red and burning eyes, increased secretion of tears, swelling and redness of eyelids or eyelid margins. Moreover, HYLO® FRESH contains sodium hyaluronate. Sodium hyaluronate increases the viscosity of the moisturizing solution so that it becomes more viscous than water. Thus, it stays longer on the ocular surface. The application of the eye drops soothes irritated eyes and eyelids and provides a tear-like moistening. With it all symptoms are treated that are caused by drying of the surface of eye, e.g. because not enough natural tears are produced by the body. HYLO® FRESH is used for the moistening of the eyes even when wearing contact lenses.

Being preservative-free, HYLO® FRESH is very well tolerated. Moreover, HYLO® FRESH is phosphate-free, so that possible complications due to the formation of precipitations in the cornea are avoided.

How should HYLO® FRESH be applied?

Only use HYLO® FRESH if the sealing of the folding carton was intact before the first use (☺).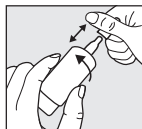

Step 1:

Remove cap before use.

For this purpose, hold the cap just over the bottle. With your other hand, pull away the bottle from the cap with a rotary movement.


Before the first application please turn the HYLO®

FRESH bottle with its tip down and press onto the bottom of the bottle until the first drop appears at the nozzle. Now the bottle is ready for further use.

Step 2:

Hold the bottle upside down with the thumb on the shoulder of the bottle and the other fingers on the bottom of the bottle.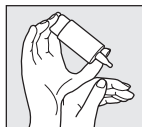


Step 3:

Support the hand holding the COMOD®-bottle as shown with the other hand.

Step 4:

Lean your head back slightly, gently pull down your lower eyelid and apply a strong and quick pressure on the middle of bottom of the bottle. This activates the mechanism for the release of one drop. Because of the special COMOD® valve system, size and speed of each drop are the same independently on how much pressure is applied. Slowly close your eyes to allow the fluid to spread evenly on the eye surface.

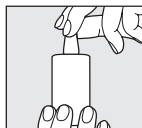Step 5:

Replace cap immediately after use. Take care that the dropper tip is dry.

When applying the fluid please avoid any contact between the tip of the bottle and your eye or your skin.

The COMOD-bottle releases 10 ml of solution which is equivalent to approximately 300 drops. For technical reasons a small amount of solution remains in the bottle at the end of usage.

The duration of use of HYLO-FRESH is not limited.

Only one person should be treated with one particular bottle of HYLO® FRESH.

What else do you need to know?

If you use any other eye drop medication please allow at least 30 minutes between the applications. HYLO® FRESH should always be used last. Eye ointments should, however, always be administered after using HYLO® FRESH.

Because it is preservative-free, HYLO® FRESH eye drops is well tolerated even when used over extended periods of time. In very rare cases hypersensitive reactions like burning or excessive tearing were reported which stopped immediately when the use of HYLO® FRESH was discontinued.

To avoid incompatibility reactions that are possible due to interactions between contact lens care products and HYLO® FRESH, we recommend to wait for about 30 minutes after inserting contact lenses before the eye drops are applied.

According to existing knowledge HYLO® FRESH can be used in children, pregnant and breast-feeding women.

How often and for how long can you use HYLO® FRESH?

HYLO® FRESH should be used according to your condition. Generally you instil one drop of HYLO® FRESH three times a day into each eye. If necessary it can also be used more frequently. If you use HYLO® FRESH more frequently (e. g. more than 10 times per day) please consult your eye doctor. HYLO® FRESH is suitable for long-term treatment. Nevertheless, as with all complaints, you should consult your eye doctor when your complaints persist.

When should you not use HYLO® FRESH?

HYLO® FRESH should not be used if you are hypersensitive to one of the ingredients.

What you need to know about storing HYLO® FRESH?

Do not store above 25 °C.

HYLO® FRESH can be used for 6 months after opening.

Do not use beyond the expiry date (☞).

Keep out of the reach of children.

The following packaging sizes are available:

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Phone: +43 676/842 799 100, Fax: +43 125 330 339 113

email: info@ursapharm.at

Your opinion about our product is of great importance for us!
We would be pleased to receive your feedback via post, fax (no. +49 6805/9292-222)
or via www.ursapharm.info/hylofresh.

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URSAPHARM Arzneimittel GmbH, Industriestraße, D-66129 Saarbrücken