# MONTESOL Nasenspülung

**DE Gebrauchsinformation**

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**EN Instructions for use**

Please read these instructions carefully before using the product for the first time.

Keep these instructions for use in a safe place.

**What does PARI MONTESOL nasal rinse contain?**

PARI MONTESOL nasal rinse with dexpanthenol is a medical product for thoroughly rinsing and cleaning the nose. The moistening of the nasal mucous membrane by the natural brine from Bad Reichenhall is supplemented with the caring and protective effect of dexpanthenol.

Composition:

Natural brine from Bad Reichenhall (25% – 27%)	99.1%
Dexpanthenol	0.9%

The bottle of PARI MONTESOL nasal rinse contains 250 ml concentrated solution for the preparation of 30 rinses.

**When is PARI MONTESOL nasal rinse used?**

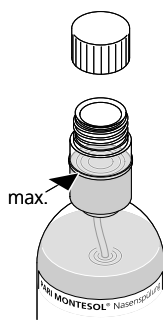
PARI MONTESOL nasal rinse is used to rinse the nose. This is helpful in the intensive cleaning and moistening of the nose.

Rinsing of the main nasal cavity is employed in the following situations:

- in frequently occurring colds
- in susceptibility to cold infections
- in dry nose and nasal calculus
- in a very dusty and contaminated environment
- in pollen dispersal and allergic persons
- concomitant therapy in chronic inflammation of the nose and the paranasal sinuses
- concomitant therapy after surgery in the region of the nose and the paranasal sinuses

**How and how often is PARI MONTESOL nasal rinse used?**

To prepare a nasal rinse solution, the dosing chamber in each instance is filled with lukewarm water to 250 ml and this is used to rinse the nose.

- Remove the screw cap from the bottle and squeeze the bottle until the level of concentrated brine reaches the dosage line in the dosing chamber. If too much saline solution enters the dosing chamber, it will run back again when you stop squeezing the bottle.
- Pour this dosed unit into the PARI MONTESOL nasal douche and then fill it with lukewarm water up to a quantity of 250 ml.

The nasal rinse is now ready to use.

The fresh water that is used in preparation must at least be of drinking water quality. It may be necessary to use boiled or sterile water to treat particular illnesses such as cystic fibrosis or directly after surgery.

The PARI MONTESOL nasal rinse

- must not be used undiluted.
- is not intended for ingestion.

We recommend using the PARI MONTESOL nasal douche for rinsing. Rinsing from the palm of the hand is not recommended for hygienic reasons.

In addition, follow the instructions for use of the PARI MONTESOL nasal douche.

Nasal rinses can be performed up to 3x daily with 250 ml of solution in each instance (dosage here: 1 dosed unit) up to 500 ml (dosage here: 2 dosed units). Rinsing is usually recommended 1x daily. During long-term rinsing therapies (several months or years), rinsing treatments performed once every 2-3 days also have been effective.

**What else must be considered when preparing a nasal rinse solution?**

It is important to observe the correct concentration of the rinsing solution, since rinsing solutions that are too high or too low in concentration can cause irritations or side effects.

The concentrated PARI MONTESOL nasal rinse when dissolved according to instructions always yields an isotonic salt solution that matches the mineral content of the blood.

Please only use rinsing solutions that have been prepared with drinking water immediately after preparation, since PARI MONTESOL nasal rinse does not contain preservatives and the prepared solution can become contaminated during storage.

**When should PARI MONTESOL nasal rinse not be used?**

PARI MONTESOL nasal rinse should not be used if, as a result of injuries to the roof of the nose and the walls of the paranasal sinuses, direct connections exist to the fluid compartments of the brain. PARI MONTESOL nasal rinse should not be used during nasal bleeding.

If the soft palate is paralyzed, PARI MONTESOL nasal rinse should not be used since a risk of choking exists. If the concentrated brine comes in contact with an eye, the eye should be rinsed with a copious amount of water.

#### How long can PARI MONTESOL nasal rinse be used?

PARI MONTESOL nasal rinse can be used for an unlimited amount of time, as long as the rinsing solution is always prepared in the correct concentration.

#### What adverse side effects can PARI MONTESOL nasal rinse have?

In very rare cases when used as directed, it is possible for mild irritations such as a tingling and burning sensation of the nose to occur.

#### Notes:

Because of the brine saturation (25-27%), it is possible to completely dispense with preservatives which can damage the nasal mucous membrane. PARI MONTESOL nasal rinse should not be used after the expiration date on the package has expired or any longer than 6 weeks after opening.

Please always store out of the reach of children.

**Caution:** contains small parts which can be swallowed!  
Always keep the PARI MONTESOL nasal rinse out of children's reach.

#### Pack sizes:

The bottle of PARI MONTESOL nasal rinse contains 250 ml concentrated solution for the preparation of 30 rinses.

#### Storage:

Always keep the bottle tightly closed.

Store out of direct sunlight and freezing temperatures. Do not store above 30°C!

#### Explanation of symbols:

Please follow the instructions for use!

Production batch

Expiry date

Information as at:

August 2007


Manufacturer:

PARI GmbH

Moosstraße 3

D-82317 Starnberg

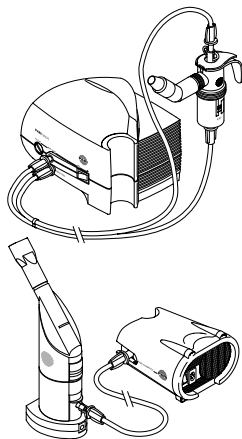Made in Germany

Ask your doctor or pharmacist about other effective treatment options:

**PARI SINUS** - The efficient inhalation for acute and chronic inflammation of the paranasal sinuses.

**PARI SOLE N** - Soothing moist and warm inhalation for disorders in the nose, mouth, neck, throat and larynx area.

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