Wund spray

D Gebrauchsinformation

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VS 11/11

GB Information on application

For users: please read the following instructions for use carefully as they contain valuable information about application of pharmaCUR® Wound Spray.

pharmaCUR® Wound Spray

Characteristics

pharmaCUR® Wound Spray is an odourless, colourless spray for antiseptic cleansing and moistening of acute and chronic wounds and of burns. It has been especially developed as first aid for wounds. pharmaCUR® Wound Spray cleanses the wound, protects it from infection and actively helps wound healing.

Application of pharmaCUR® Wound Spray is pain free, irritant free and therefore is also suitable for children, people with allergies and diabetics. Application on mucous membranes has also proved successful.

The ingredients in pharmaCUR® Wound Spray are used successfully in clinical practice, are safe for dermatological use and are considered the first choice in treatment of wounds*. pharmaCUR® Wound Spray has a very broad microbial spectrum (against bacteria, viruses and fungi) and also provides protection against multidrug resistant pathogens (MRSA).

Areas of application

To cleanse and prevent infection in:

- acute wounds: e.g. gashes, cuts, grazes, postoperative wounds; also on mucous membranes
- chronic wounds: e.g. pressure ulcers and venous leg ulcers
- wounds subsequent to 1st and 2nd degree burns

Instructions for use

Cleansing of wounds

Spray the wounds generously with pharmaCUR® Wound Spray in order to remove any contamination, blood and bacteria from the wound area. In the case of increased contamination, the application can be repeated several times.

The polyhexanide betaine complex contained in pharmaCUR® Wound Spray also thoroughly cleanses wound areas that are hard to reach, thus preventing any infection in the wound. This creates ideal conditions for the wound to heal without difficulty.

In order to prevent the wound from becoming contaminated again, protect the wound with a plaster or dressing.

Tip

After first aid using pharmaCUR® Wound Spray, we recommend the use of pharmaCUR® Wound Healing Gel with good adhesion, which is based on the principle of moist wound healing, in order to encourage healing of the wound more intensively. Cover the wound with a plaster or dressing afterwards.

Changing dressings

Encrusted plasters, wound dressings or bandages that are difficult to remove should be sprayed generously with pharmaCUR® Wound Spray until they are fully saturated. After a short time, these can be removed without causing irritation or harm to the wound surface.

You can use pharmaCUR® Wound Gel to prevent the dressing material from adhering to the wound.

Restrictions for use

Do not use pharmaCUR® Wound Spray around the auditory canal or the eyes. Careful thought should be given to use of the product during pregnancy and while breastfeeding.

If in doubt medical advice should be obtained. To date there is no evidence of harmful effects when used during pregnancy and while breastfeeding. As the ingredients are not absorbed into the system, it is not expected that they will be passed into the breast milk.

General safety instructions

pharmaCUR® Wound Spray should not be used in the case of allergies to one of the ingredients.

Do not use at the same time as other products for cleaning wounds. For external use only, do not swallow.

Composition

Aqua purificata, 0.1% undecylenamidopropyl betaine, 0.1% polyaminopropyl biguanide (polyhexanide)

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