## Breastmilk Bottles

Ideal to collect, store and freeze breastmilk

## Cleaning and Hygiene

## Important

- Use drinking water quality for any of the following described cleaning procedures.
- Clean parts immediately after use to avoid dry up of milk residues and to prevent from growth of bacteria.

## Prior to first use and once daily

Disassemble, wash and rinse all parts.  
Place parts in a saucepan of sufficient capacity.  
Fill with cold water until the parts are well covered.  
 Optionally, a tea spoon of citric acid may be added to avoid a build-up of limescale.  
Heat it up and allow to boil for 5 minutes.

Alternatively, the disassembled parts can be sanitized in Medela Quick Clean Microwave Bags or in top rack of dishwasher or steam-sterilized.

## After each use

Disassemble and wash all parts in warm, soapy water.  
Rinse in clear water and allow parts to air-dry on a clean towel.

## Warning

- Keep all components not in use out of the reach of children.

## Caution

- Avoid touching the inside of bottles and lids after cleaning.

## Storing / thawing breastmilk

(For Healthy Term Babies)

## Freshly expressed breastmilk

Room Temperature	Do not leave at room temperature
Refrigerator	3 – 5 days at approx. 4°C
Refrigerator Freezer Compartment	6 months at approx. -16°C
Deep Freezer	12 months at approx. -18°C

## Thawed breastmilk (previously frozen)

Room Temperature	Do not store
Refrigerator	10 hours
Refrigerator Freezer Compartment	Never refreeze
Deep Freezer	Never refreeze

- When freezing do not fill bottles more than 3/4 full to allow space for expansion.
- Label bottles with the date of pumping.
- Do not store breastmilk in the door of the refrigerator.
- In order to preserve the components of the breastmilk, thaw in the refrigerator overnight or by holding the bottle under running warm water (max. 37°C).
- Gently shake the bottle to blend any fat that has separated.
- Do not thaw frozen breastmilk in a microwave oven or in a pan of boiling water (danger of burning).

Material: Polypropylene  
Disposal: According to local regulations

Does not contain Bisphenol-A, DEHP (Phthalate) or Latex.  
Conforms to EN 14350.

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